

Travel Lite Massage Table w/carry case

OWNER'S MANUAL

MODEL #9100



The specifications of this product may vary from this photo, subject to change without notice.

**To request for product service and
order replacement parts, please call
our customer service department at:
1-866-924-1688**

8:00 am - 5:00 pm (PST) Monday through Friday

Email: service@paradigmhw.com

**If the product has major defects which prevent it from
functioning properly, please return it to the store of
purchase within the period allowed by the store.**

When request for service, please have the following information ready:

- 1. Owner's manual**
- 2. Model number**
- 3. Parts number and description**
- 4. Date of purchase**

TABLE OF CONTENTS

Precautionary Instructions	2
Overview	3
Parts List	4
Setting up Your Table	5
Adjustment	6
Storage & Fold up the Table	7

PRECAUTIONARY INSTRUCTIONS

- (1) Maintain the table periodically. Make sure all screws are secure and the cable has not deteriorated or frayed. Make sure there are no cracks in the wood structure.**
- (2) Do not overload the table beyond the recommended weight limit. This could result in serious injury to the user.**
- (3) Do not use alcohol based cleaners when cleaning the vinyl. This could cause damage to the vinyl material. Use warm soapy water and wipe dry.**
- (4) Make sure when using the table that all four legs are at the same level. Using the legs at different levels can cause the table to be unstable and cause injury.**


WARNING !

Maximum weight capacity

Static weight : 1600 LBS / 726KGS

Working weight : 400 LBS / 181KGS

OVERVIEW

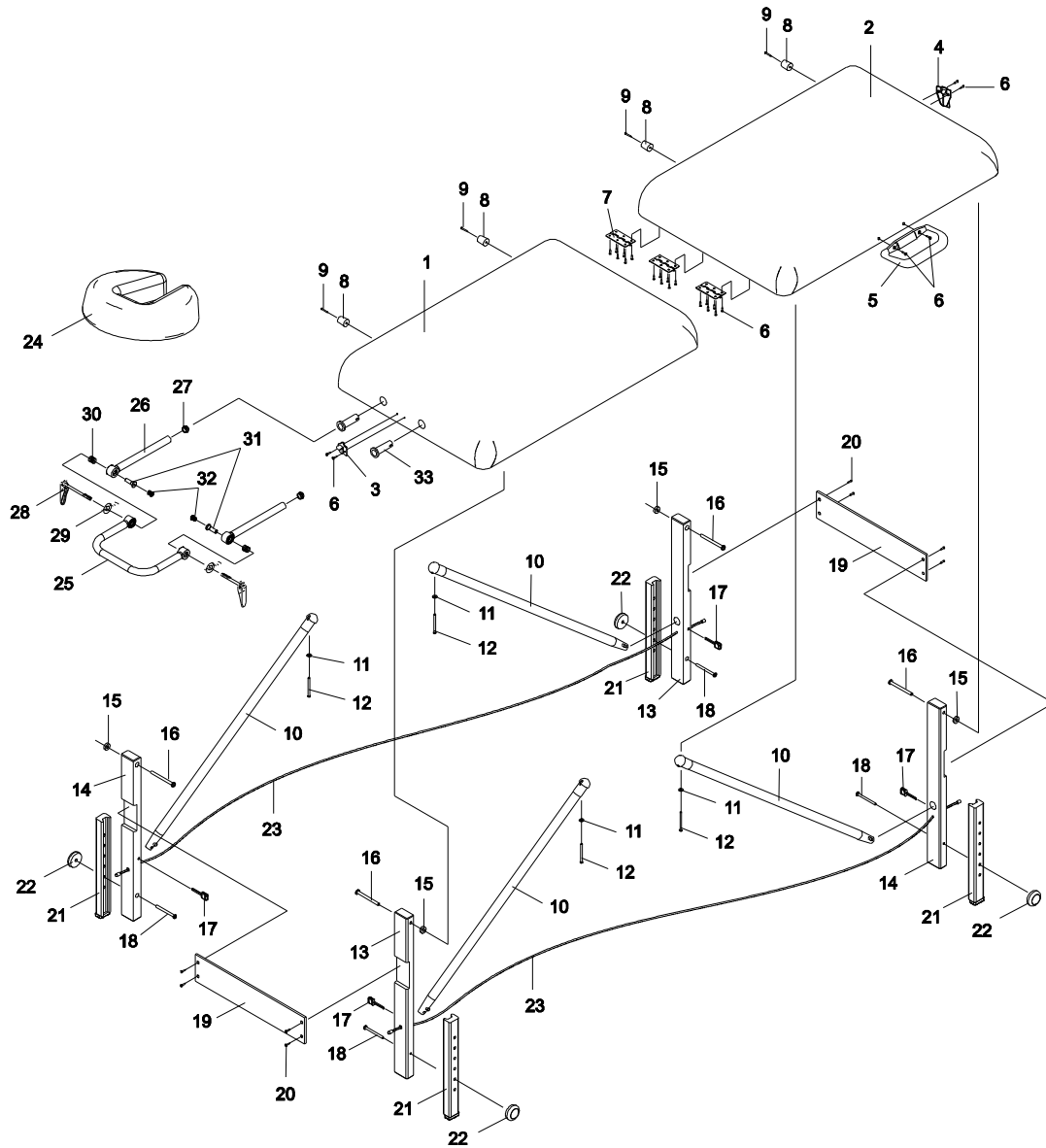

PARTS LIST


Part #	Description	Quantity	Part #	Description	Quantity
001	Front Main Frame	1	018	M6x63.5mm Bolt	4
002	Rear Main Frame	1	019	Cross Brace	2
003	Upper Buckle	1	020	M4x22mm Self Tapping Screw	8
004	Lower Buckle	1	021	Lower Leg	4
005	Handle	1	022	Knob	4
006	M4x29mm Self Tapping Screw	24	023	Short Cable	2
007	Hinge	3	024	Face Cushion	1
008	Rubber Foot	4	025	Head Rest	1
009	M4x38mm Self Tapping Screw	4	026	Head Cradle Tube	2
010	Strut	4	027	Head Cradle Plug	2
011	Washer Ø12xØ5.1x1.0T	4	028	Lock Handle	2
012	M4.5x51mm Bolt	4	029	Lock Handle Washer	2
013	Upper Leg (A)	2	030	Spring	2
014	Upper Leg (B)	2	031	Lock Handle Bushing	2
015	Rondelle PVC	4	032	Aluminum Nut	2
016	M8x76mm Bolt	4	033	Bushing	2
017	Safety Locking Pin	4			

SETTING UP YOUR TABLE

1. Unlace the buckle.




2. Open your table so that it is standing on the Rubber Foot side at the angle shown in the diagram #2.

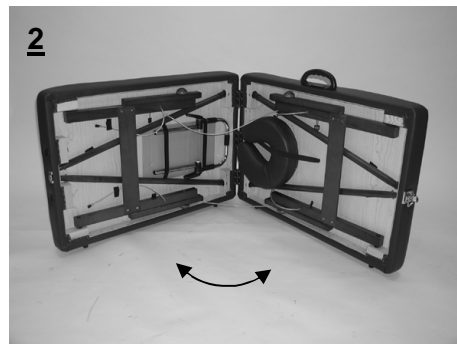


3. Pull out the left and right pair of Legs and make sure the Short Cables are secured in the leg holes.

WARNING: DO NOT USE THE TABLE IF THE SHORT CABLES ARE NOT SECURED IN THE LEG HOLES!

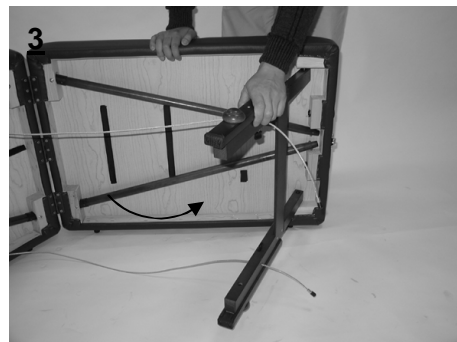

4. Insert the end of Strut into the hole of Upper Leg and then push in the Safety Locking Pin so that it locks the Strut in position. Do the same steps for another three Struts. Please make sure they are locked in position. Please refer to the diagrams 4 and 4-1.

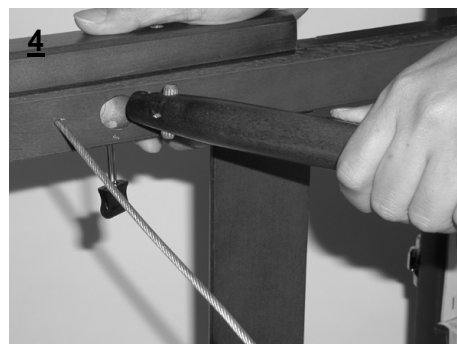

WARNING: FAILURE TO LOCK THE PINS COULD RESULT IN TABLE COLLAPSING AND LEAD TO SERIOUS INJURY!

5. Place the Face Cushion on the Head Rest with the designated velcro stickers, and then insert the Head Rest into the Bushings of Front Main Frame.



Placing the Table in Carry Case:

Unzip the Carry Case and place it open on the floor. Place the table into the Case and zip up the Case at both ends.

The Carry Case should be used when transporting or storing the table, to help protect the table from damage. When not in use, store the table in a cool dry place, and away from direct sunlight. Do not expose the table to hot or cold temperature, or moisture.



ADJUSTMENT

Angle of Head Rest Adjustment:

Adjust the Aluminum Nut first to ensure proper contact between the two teathed surface, then securely fastening it by pressing down the Lock Handle. Please refer to the small diagrams 1 and 2.

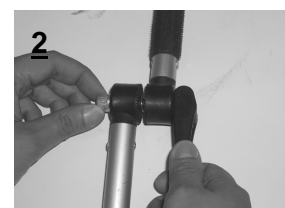
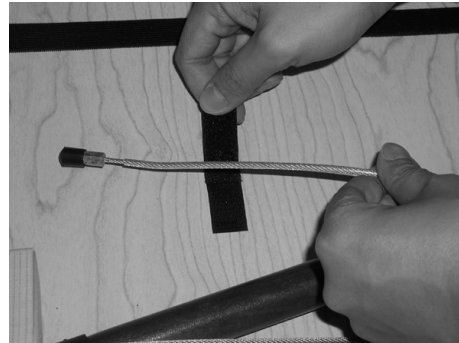


Table Height Adjustment:

You can adjust the height of your table by removing the Knob from each leg as the table stands, adjust each leg to the desired position, and replace the Knobs. Make sure all legs are adjusted to the exact same height position before using.

STORAGE

Place the Short Cable end on the bottom of table with the designated velcro sticker. Do the same step for another three Short Cable ends.

**FOLD UP THE TABLE**

1. Lay the table on its side and fold at an angle.
2. Completely fold the Legs flat under the table.
3. Make sure the Short Cables, Face Cushion, Head Rest, and Struts are inside the table before closing it.